



MULTI ACTION 5

**BACTERICIDE
VIRUCIDE
FUNGICIDE
DIN#02264021**

LIQUID DISINFECTANT FOR HARD SURFACES

GUARANTIE / GUARANTEE:

Alkyl (C ₁₄ 50%, C ₁₂ 40%, C ₁₆ 10%) Dimethyl Benzyl Ammonium Chloride.....	0.20%
Octyl decyl Dimethyl Ammonium Chloride.....	0.15%
Dioctyl Dimethyl Ammonium Chloride.....	0.06%
Didecyl Dimethyl Ammonium Chloride.....	0.09%

DESCRIPTION

MULTI ACTION 5 is a ready-to-use, highly effective cleaner and disinfectant for use on the surfaces of inanimate objects. It is especially useful to disinfect non-porous hard environmental surfaces in health care facility (hospitals, dental clinic, nursing homes) and / or a food processing plant.

APPLICATION

For cleaning medical equipment and surfaces

Apply directly to surface. Wipe surface cleaner using a paper or cloth towel or rinse surface and either wipe dry or allow to air dry. Discard dirty towel.

For disinfecting medical equipment and surfaces

Apply directly to precleaned surface, thoroughly wetting area to be disinfected. Allow surface to remain wet for 10 minutes at room temperature (20°C). Follow by wiping surface using a clean paper or cloth towel; or rinse and either allow surface to air dry or wiped rinsed surface dry using a clean paper or cloth towel. Discard dirty towel.

For disinfecting surfaces in food processing establishments:

To Disinfect: Apply directly to precleaned surface, thoroughly wetting area to be disinfected. Allow treated surfaces to remain wet for a minimum of 10 minutes in a single application. Can be applied with a mop, sponge, or cloth as well as coarse (trigger) spray. For food plant use, food contact surfaces should be rinsed with abundant potable after disinfection.

PROPERTIES

Appearance: Clear liquid

Odor: Alcohol